

care*fix*[®]
Creating functional comfort




Promotes the healing process by controlled compression



Easy to apply, allowing easy examination

Provides optimal comfort and skin-friendliness

SOPHIA
Post-Op Bra
Compression Bra/Surgical Vest

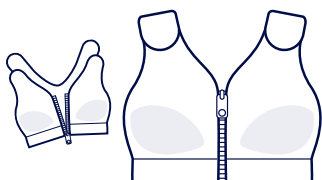

TYTEX[®]

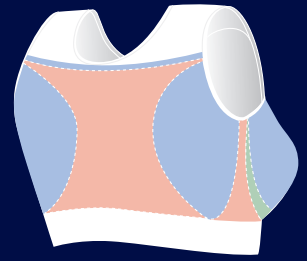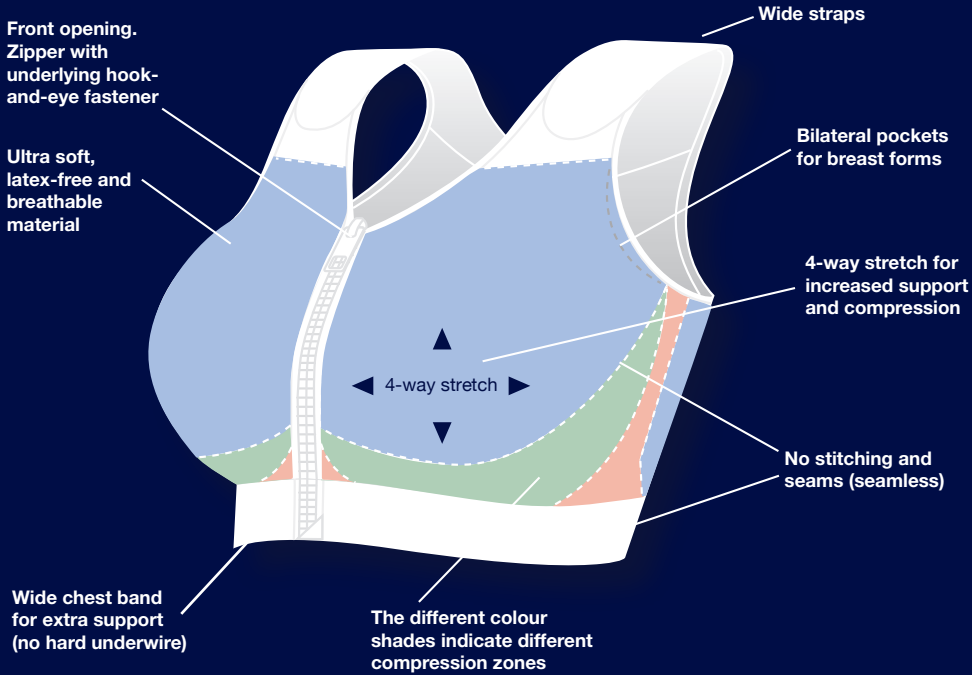

Compression zones at the back



Fully openable for easy application (both chest and shoulder straps)

The Sophia Post Op bra/Surgical Vest offers optimal stabilisation and immobilisation of the operated breast. It promotes the healing process by controlled compression on the scar tissue.

With the front opening and adjustable Velcro straps it is easy to apply the bra for both nursing staff and patient, and it allows easy examination of the surgical site.

The 100% seamless knitting increases comfort for the patient, prevents pressure marks and tissue irritation, resulting in maximum skin-friendliness.

Indications:

Medical device for post-surgical use after breast surgery (with pockets for breast puffs).

Precautions:

If you have any questions regarding usage and correct fitting, please consult your doctor or nurse. Make sure that any bandages and/or drainage tubes underneath the bra are located correctly to avoid pressure points and/or obstructed flow from the drainage tubes. The bra must be removed during radiotherapy due to metal parts. If any unexpected discomfort, wound irritation or similar occur, please consult your doctor or nurse. NOTICE: For breast augmentation a stabilizer band for breast implants may also be required. Follow the instructions from your doctor or nurse.

Recommended sizes

Band Size	EU	60 cm	65 cm	70 cm	75 cm	80 cm	85 cm	90 cm	95 cm	100 cm	105 cm	110 cm
	F/B/E	75	80	85	90	95	100	105	110	115	120	125
	US/UK	24 inch	26 inch	28 inch	30 inch	32 inch	34 inch	36 inch	38 inch	40 inch	42 inch	44 inch
Cup Size	A											
	B											
	C											
	D											
	DD											
	F											
	S	M	L	XL	XXL							

Available in white and black.

Composition:

- 78% Cotton
- 15% Polyamide
- 7% Elastane

Washing instructions:



Coloured 40°C/105°F



White 60°C/140°F

Close zipper and Velcro before washing. Wash with similar colours and in a washing bag. Using chlorine-based bleach or chlorine-based disinfection will decrease durability and function.

For further information please contact:

Dealer: